D/GB

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Translation of the original user manual

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CE-Declaration of conformity

The company AAT Alber Antriebstechnik GmbH herewith declares that the s-max sella is conform to the current regulations of the European Union, EEC Guidelines 2006/42/EC, 2007/47/EC and 93/42/ECC. This declaration becomes void if the s-max sella has been altered without the consent of AAT Alber Antriebstechnik GmbH.

Signature of the manufacturer:

Markus Alber
(Managing shareholder)

The stair climber s-max sella has been successfully tested according to the standards DIN EN 12182 and the applicable parts of ISO 7176 at the Technical University of Berlin, Berlin-Cert.

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Table of contents

1	Symbols and safety instructions	.07
2	Description of the product.	.10
2.1	Features of the device.	.10
2.2	Information to the devices.	.11
2.3	Appropriate use.	.11
2.4	Technical data	.14
2.5	Dimensional sketch D1353	.16
2.6	Dimensional sketch D1603	.17
2.7	Scope of delivery	.18
2.8	Accessories	.18
3	Assembly	.20
3.1	Handle unit	.20
3.2	Battery-pack	.22
4	Initial operation	.23
4.1	Fold out the handle/seat unit	.23
4.2	Armlehnen einsetzen	.24
4.3	Adjust handle	.25
4.4	Turn on the device	.26
4.5	Neutral position	.27
4.6	Check the safety brakes.	.28
4.7	Single step mode	.29
4.8	ComfortStep	.29
4.9	Check capacity of the batteries	.30
5	Operation	.31
5.1	Occupant taking a seat	.31
5.2	Swivel out or remove the arm rests	.32
5.3	Fold out the foot rest	.32

5.4 Operation on flat ground	.33
5.5 Adjust the handles	.34
5.6 Tilt the stair climbing system	.35
5.7 Operation on a slope	.37
5.8 Climbing - upstairs	.40
5.9 Climbing - downstairs	.43
5.10 Lay the s-max down on the stairs.	.45
5.11 Parking	.46
5.12 Fold up the handle/seat unit	.47
5.13 Transportation	.48
 6 Care and maintenance	 .49
6.1 Batteries	.50
6.2 Exchange the electrical fuses	.54
6.3 Check the lining of the brakes	.56
6.4 Check the rollers on the climbing feets	.56
6.5 Cleaning/Desinfection	.57
6.6 Recycling	.58
 7 Malfunction and trouble shooting	 .59
7.1 Malfunction	.59
7.2 Trouble shooting	.62
 8 Warranty and liability	 .63
8.1 Warranty	.63
8.2 Liability	.64

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1 Symbols and safety instructions

WARNING - Can lead to serious injuries or death!

CAUTION - Can lead to minor injuries!

NOTE - Can lead to material damage!

- » Before initial operation, read the user manual and take note of the warning and safety instructions.
- » Before initial operation, check the electronic functions, the safety brakes and the lining of the climbing feet.
- » Before initial operation, store the s-max sella in a non-freezing environment at least for one hour.
- » Observe technical data.
- » The approved total weight and the max. weight of the occupant may not be exceeded.
- » Operation only by one trained person. Minors of 16 and above may operate the device only under supervision. This person should be physically and intellectually capable of handling the s-max sella safely.
- » Always operate with both hands.
- » Practice driving without a occupant and in single step mode.
- » Wear non-slip footwear.
- » For the transportation of persons, not of objects.
- » Secure occupants who cannot sit on their own with a belt.

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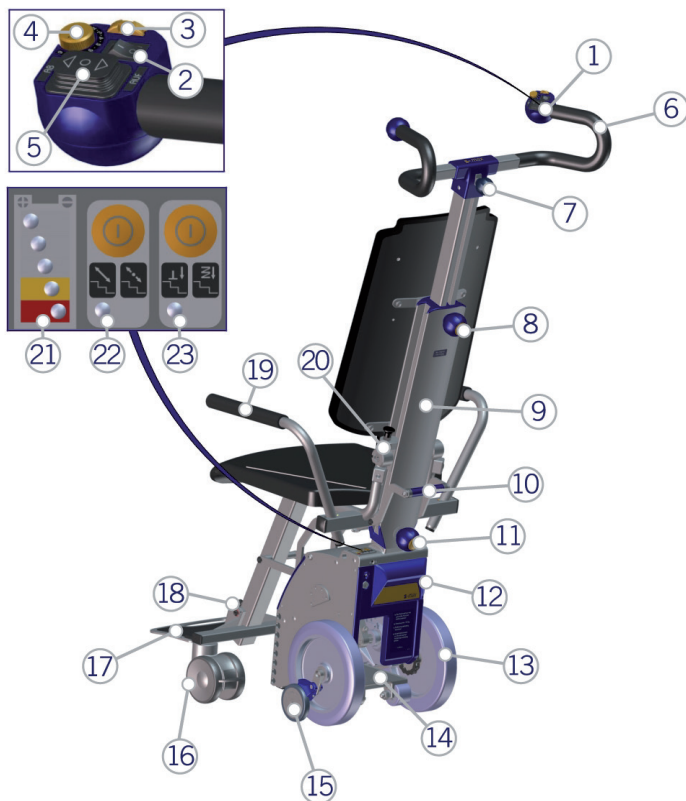
①
 ②
 ③
 ④
 ⑤
 ⑥
 ⑦
 ⑧
 ⑨
 ⑩

Control unit
 ON/OFF switch
 Safety chip for immobilizer
 Speed control
 UP/DOWN switch
 Handle
 Locking screw for horizontal adjustment
 Locking screw for height adjustment
 Column
 Release lever

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2.2 Information about the devices

This user manual is an all-purpose manual for the devices s-max sella D1353 and s-max sella D1603. The only differences between the stairclimbers are the technical details as well as deviations in optics.

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2.3 Appropriate use

With the stair climber s-max sella you are able to transport one person safely and effortlessly up- and downstairs. Driving on slopes is also possible.

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2.4 Technical data

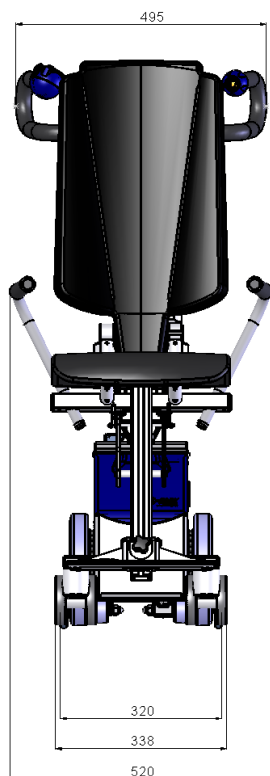
Technology	s-max sella D1353 (135 kg Version)	s-max sella D1603 (160 kg Version)
Measurements	Height = min. 1150 mm, max. 1550 mm Width: Handle = 495 mm, Climbing unit = 320 mm Total = 520 mm Depth: Handle = 212 mm Total = 890 mm	Height = min. 1150 mm, max. 1550 mm Width: Handle = 495 mm, Climbing unit = 338 mm Total = 520 mm Depth: Handle = 212 mm Total = 890 mm
Weight	Climbing unit = 15,8 kg Handle/seat unit = 14,8 kg Battery-pack = 3,0 kg Both arm rests = 1,2 kg Charger = 0,2 kg Total weight = 35,0 kg	Climbing unit = 16.9 kg Handle/seat unit = 14.8 kg Battery-pack = 3.0 kg Both arm rests = 1.2 kg Charger = 0.2 kg Total weight = 36.1 kg
Total weight approved	170 kg	195 kg
Max. weight of the occupant	135 kg	160 kg
Climbing speed ¹⁾	5 - 20 steps / minute, continually adjustable	8 - 23 steps / minute, continually adjustable
Range with one charge of the batteries on average load of 75 kg ¹⁾	approx. 26 levels (equals 390 steps)	approx. 20 levels (equals 300 steps)
Range with one charge of the batteries on maximum load ¹⁾	approx. 11levels (equals 165 steps)	approx. 9 levels (equals 135 steps)
Maximum height of step	22 cm 23.5 cm (extended)	22 cm 23.5 cm (extended)
Necessary step width (straight stairs)	Width s-max sella + 100 mm	Width s-max sella + 100 mm

Min. platform size (U-shaped stairs)	Depends on the attendant's height	Depends on the attendant's height
Batteries	2 x 12 V / 3,3 Ah sealed lead acid	2 x 12 V / 3.3 Ah sealed lead acid
Direct current motor	24 V / 200 W	24 V / 275 W
Operating force UP/DOWN-switch	3-8 N	3-8 N
Sound pressure level	< 70 dB	< 70 dB
Storing/operating temperature,device	0° C bis +60° C / -30° C bis +50° C	0° C bis +60° C / -30° C bis +50° C
Storing/operating temperature,batteries	0° C bis +60° C / -30° C bis +50° C	0° C bis +60° C / -30° C bis +50° C
Vibrations	The weighted RMS value of acceleration the upper body is exposed to lies typically below 2.5 m/s ²	The weighted RMS value of acceleration the upper body is exposed to lies typically below 2.5 m/s ²
Sticker serial number	On the left side of the housing.	On the left side of the housing.
IP-Protection class	IPX4 protection class: Protection against splash water from all sides.	IPX4 protection class: Protection against splash water from all sides.

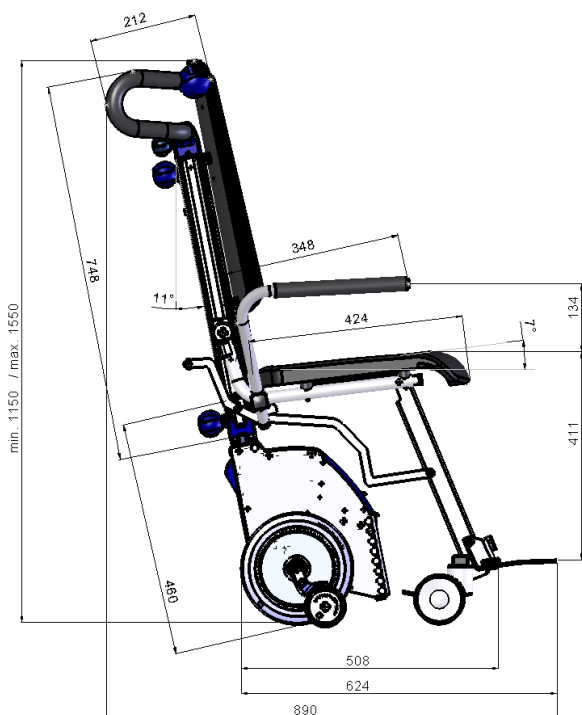
¹⁾ The climbing speed as well as the range with one charge of the batteries is reduced if the ComfortStep is activated.

In the interest of progress we reserve the right for technical changes.

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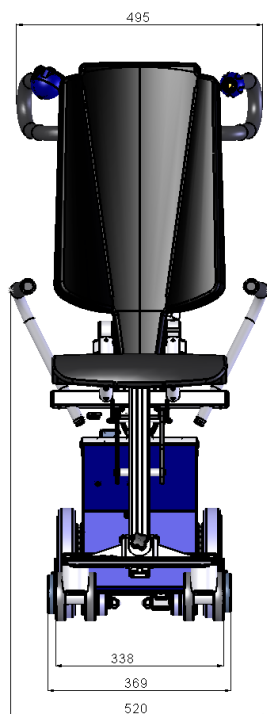
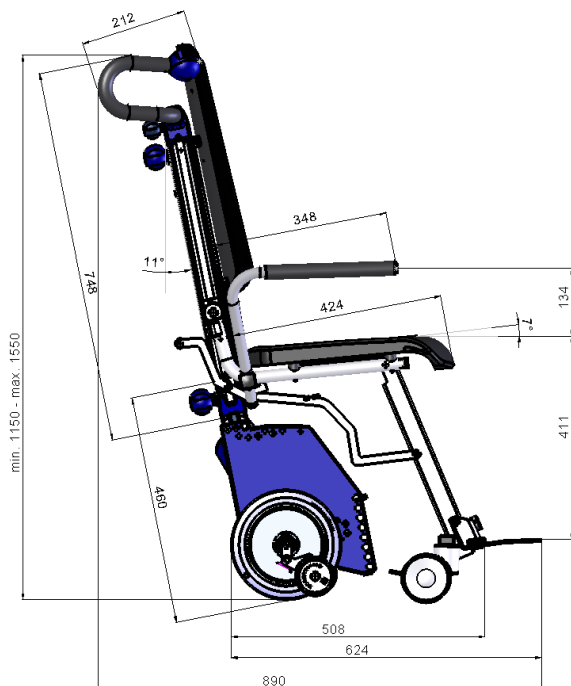
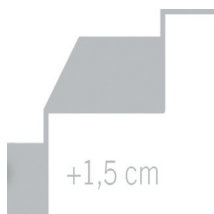
WARNING - Tighten the locking screws!

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WARNING - Only authorized AAT dealers or an AAT representative are allowed to adjust the holding device.

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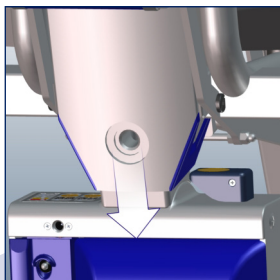
Connect the handle/seat unit with the climbing unit.

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Insert the column of the handle/seat unit (seat pointing forward) into the base on the climbing unit.

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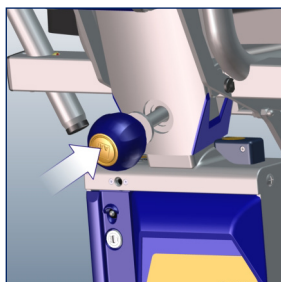
A 9-pin Sub-D socket connector located in the climbing unit's holding device makes the electrical connection.

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Secure the connection with the locking screw.

- » Please always make sure that the handle/seat unit is safely connected to the climbing unit.

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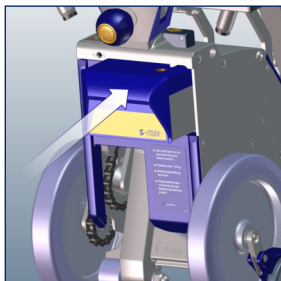
WARNING - Batterie-pack has to
be locked!

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**Attach the battery-pack.**

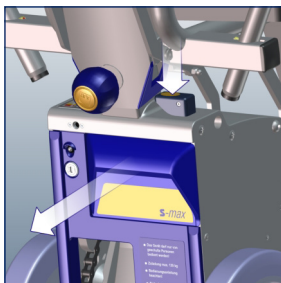
Push the battery-pack sideways into the climbing unit.

- » Once connected, it is automatically locked.

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**Detach the battery-pack.**

Unlock the battery-pack.

- » Push down the locking lever on the climbing unit and take out the battery-pack.

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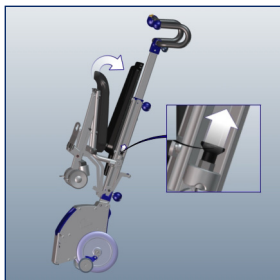
NOTE - Before initial operation,
store the s-max sella in
a non-freezing environment!

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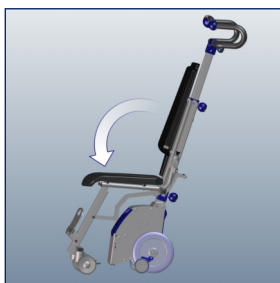
Press the seat toward the back cushion.

- » At the same time pull up the release button.

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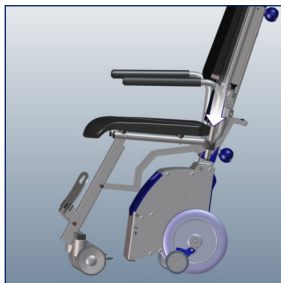
Now pull down the seat.

- » You must hear the seat click into place!

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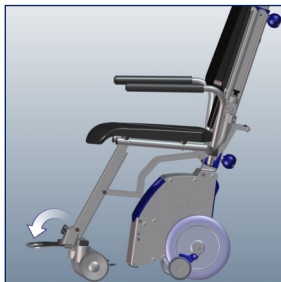
4.2 Attach arm rests

Insert arm rests on both sides.

- » Please always make sure that the stop bolts snap into their appropriate notches.

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Fold out the foot rest

- » Please fold out the foot rest to transport a person.

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WARNING - Make sure that the handle is tightly affixed!

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In very narrow stairwells or on winding stairs the handle may hit the wall.

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The handle can be adjusted horizontally.

» Release the locking screw for horizontal adjustment counter clockwise.

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Adjust the handle horizontally to the desired position.

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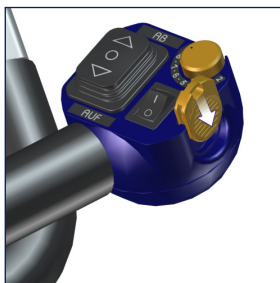
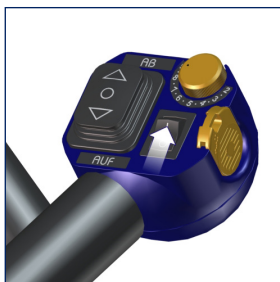
4.4 Turn on the device

Insert safety chip for the engine immobiliser.
» Safety chip must latch!

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Battery display glows.
» Single step mode is activated!
» (see chapter 4.6)

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Falsch.

4.5 Neutral position

Bring climber into the neutral position by pressing the UP/DOWN switch.

» If single step mode is activated, climber will stop automatically in neutral position.

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WARNING - Do not use the s-max sella, if the required brake effect (on one side or both sides) is not guaranteed! Contact manufacturer or authorized personnel!

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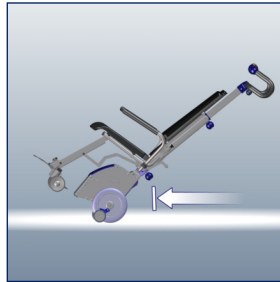
WARNING - Check safety brakes without an occupant!

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There are safety brakes located on the climbing frame on both sides

» Check the safety brakes each time before you use the s-max sella.

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Pull in the climbers to their upmost position and tilt the s-max sella backwards.

Test: s-max sella cannot be pushed forward, only backward.

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4.7 Single step mode

Deactivate / Activate.

- » LED glows = single step mode activated!

Should you choose to turn off the single step mode you can do so by pressing the appropriate button.

If the single step mode is deactivated the climbing process will be permanently continued after every step.

4.8 ComfortStep

Deactivate / Activate.

- » LED glows = single step mode activated!
- » It ensures a pleasant and comfortable climbing process.

Should you choose to turn off the ComfortStep you can do so by pressing the appropriate button.

Due to the memory function the last setting is stored. Everytime you turn on the s-max sella, the last used setting will be reproduced.

If an adjustment of the step height is necessary, please contact the manufacturer or your authorized dealer!

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Red = empty

» Charge batteries!

» Audio warning every
12 sec.!» The capacity of the
batteries is sufficient for
max. 20-35 steps

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WARNING - Operation only by one trained person!

WARNING - Practice driving without an occupant and in single step mode!

WARNING - Always operate with both hands!

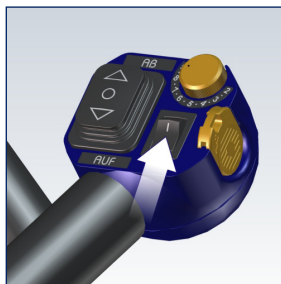
CAUTION - The arms should not protrude over the arm rests!

NOTE - Wait a few seconds after turning on the single step mode!

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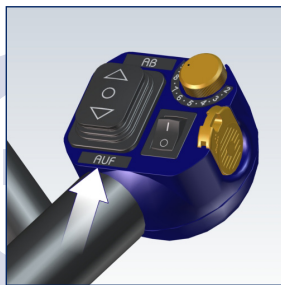
5.1 Occupant taking a seat

Turn on s-max sella.

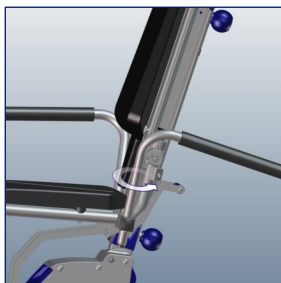
The s-max must sit securely on the ground. Press the UP/DOWN switch, direction UP.

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5.2 Swivel out or remove the arm rests

Sitting down can be facilitated by folding out the arm rests. If necessary you can remove the arm rests altogether.

» After the occupant to be transported has taken a seat you put the arm rests back into their original position.

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5.3 Fold out the foot rest

When an occupant is transported, please fold out the foot rest and, if necessary, help the person to place his/her feet on it.

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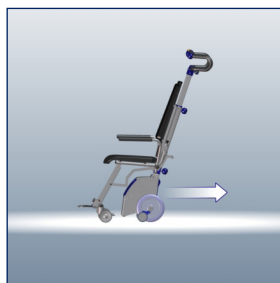
5.4 Operation on flat ground

The climber has to be retracted entirely.

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You overcome a threshold by pulling the s-max sella backwards over it.

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5.5 Adjust the handles

Release the tightening knobs counter clockwise.

Adjust the height of the handles approx. to the level of the operator's shoulder.

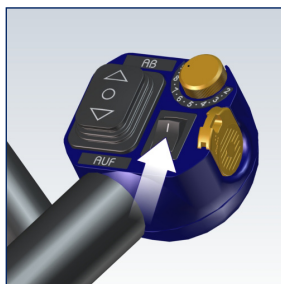
Tighten the tightening knobs again.

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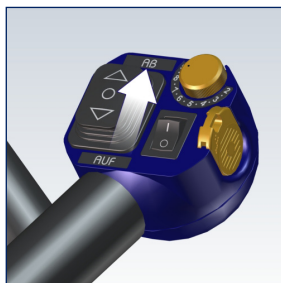
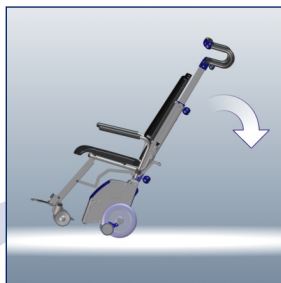
5.6 Tilt the stair climbing system

WARNING - To drive on slopes or to climb stairs you have to tilt the s-max sella.

Turn on the s-max sella by pressing the ON/OFF switch.

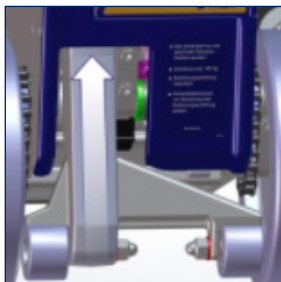
Press the UP/DOWN switch, direction DOWN.

» The climber has to lift the main wheels at least 1-2 cm off the floor.

Put your foot on the power unit's housing and tilt the s-max sella slightly towards you until you feel that the device is balanced.

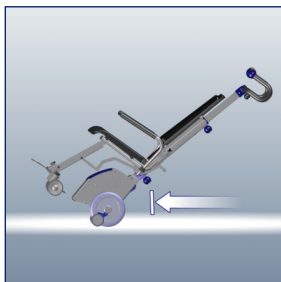
» The s-max sella is balanced if you feel neither pull nor push in the handle.

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In order to drive on flat ground, the climber has to be retracted entirely.

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If you tilt the s-max sella further, or too much, the safety brakes engage and the s-max sella can't be moved forward any more.

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5.7 Operation on a slope

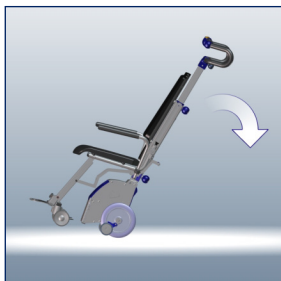
ATTENTION - For safety reasons we recommend that you ask another person for assistance when driving on a slope!

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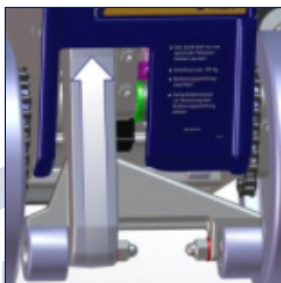
Make sure the handles are adjusted appropriately.

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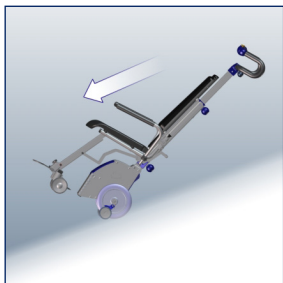
Climbers have to be retracted entirely.

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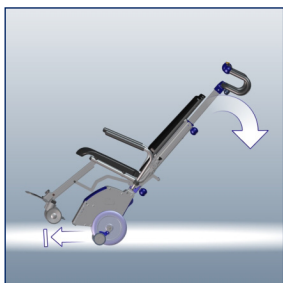
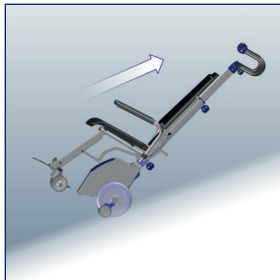
In order to slow down you need to tilt the s-max sella a bit more, so that the safety brakes are activated.

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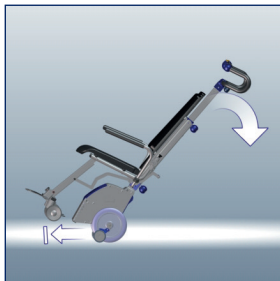
WARNING - Operation only by one trained person! A minor over 16 must be under supervision when operating the s-max. This person should be physically and intellectually capable of handling the s-max sella safely!

WARNING - Practice driving without an occupant and with single step mode!

WARNING - Before initial operation, check the safety brakes!

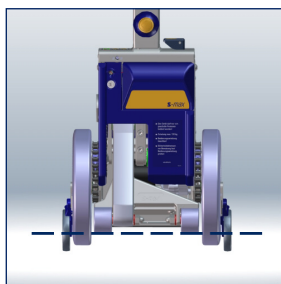
WARNING - Always operate with both hands!

WARNING - Other persons are not allowed to be in the danger area underneath the s-max sella!

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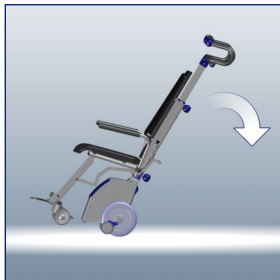
Put in the neutral position.

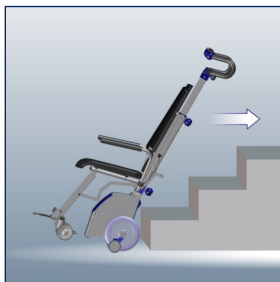
» Climbers have to be retracted entirely.

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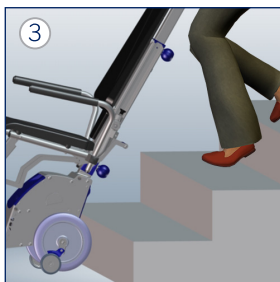
Tilt the s-max sella and
keep it in balance.

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Drive backwards all the way
to the step.

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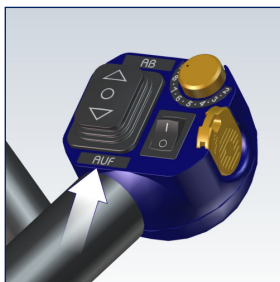
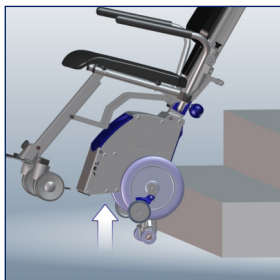
Press UP/DOWN switch, direction UP.

» If single step mode is activated, climber will stop automatically in the neutral position.

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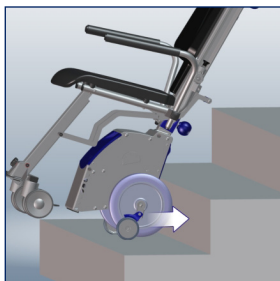
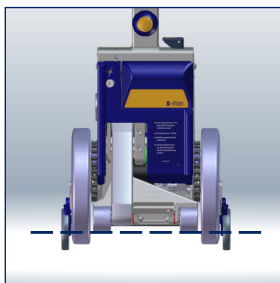
WARNING - As soon as the climbers resume the load, you feel a tension in the handles. To counteract this, briefly hold s-max lower.

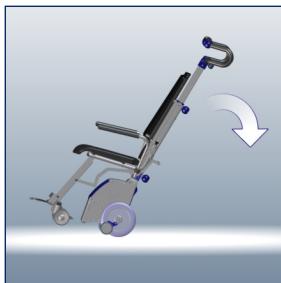
WARNING - Other persons are not allowed to be in the danger area underneath the s-max sella!

Neutralstellung einstellen.

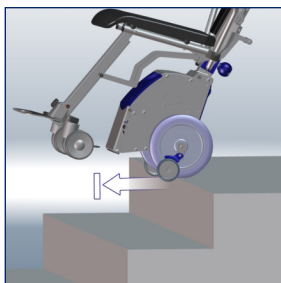
Put the s-max into the neutral position.

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Tilt s-max sella and keep it in balance.

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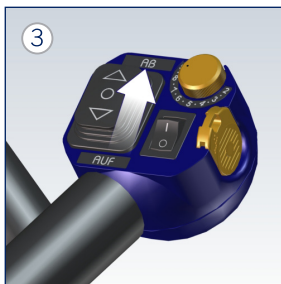
Push s-max sella forward until safety brakes grip.

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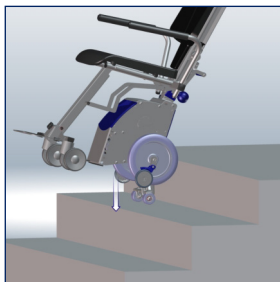
Press UP/DOWN switch, direction DOWN.

» If single step mode is activated, climbers will stop automatically in the neutral position.

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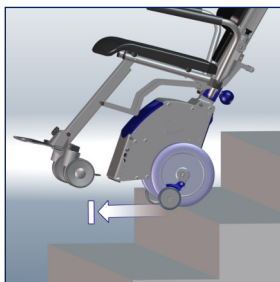
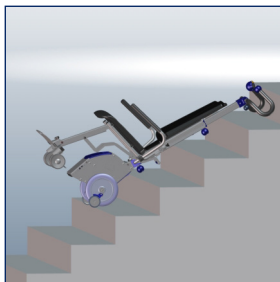
Lay down the s-max sella.

» Safety brakes prevent the device from rolling down!

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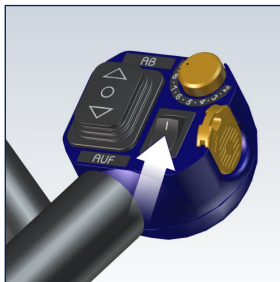
The automatic switch-off turns the s-max sella OFF during longer breaks of non-use. Turn the s-max sella OFF first and then ON.

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Put the s-max into the neutral position.
The climber has to be retracted entirely.

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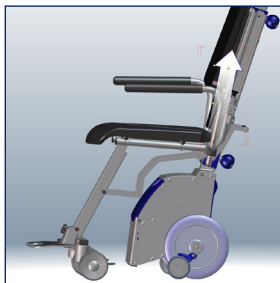
Remove the safety chip for the engine immobiliser.

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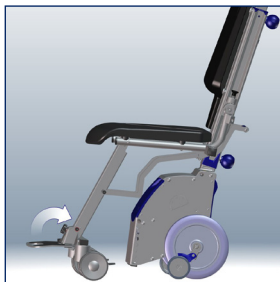
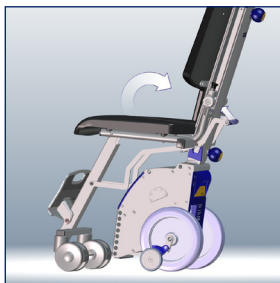
Push the release lever forward (to the column) and fold up the seat.

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Push the seat's cushion towards the back rest until you hear it snap into place.

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WARNING - Observe the weight specifications!

WARNING - Carry the climbing unit at its recessed handle as well as the battery-pack's handle!

WARNING - During transportation in vehicles, secure the s-max sella according to the relevant legal regulations!

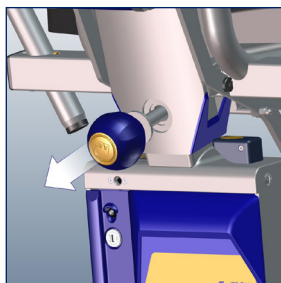
CAUTION - Turn off the s-max sella for transportation!

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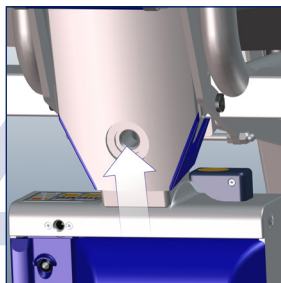
The batteries are not considered dangerous according to the IATA special guidelines A67 and GGVS Rn-Nr. 2801 A, paragraph 2! Batteries are admitted to air travel by DOT and IATA!

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Remove the handle/seat unit.
» Retighten the locking screw.

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WARNING - We recommend a safety check every two years!
Tip: We recommend a safety check, if the s-max sella goes to a new user.

WARNING - Repairs and safety checks may only be done by the manufacturer or authorized personnel!

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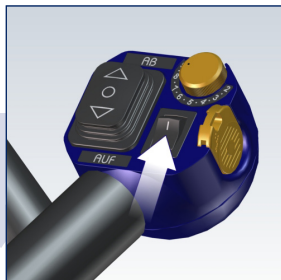
s-max sella ausschalten.

6.1 Batteries

CAUTION - Never touch the s-max sella with moist hands!

NOTE - Protect the s-max sella against humidity!

NOTE - Charge the batteries after each use!

NOTE - Protect the charger and the voltage converter from oils, grease, aggressive cleaning agents, thinners or damage!

NOTE - Follow the user manual of the charger and the voltage converter!

NOTE - A total discharge leads to a loss of the batteries' capacity and shortens their durability!

New batteries reach their full capacity after several chargings. Batteries are admitted to air travel by DOT and IATA.

The batteries are maintenance-free and rechargeable.

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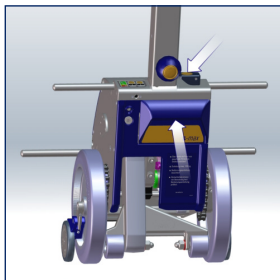
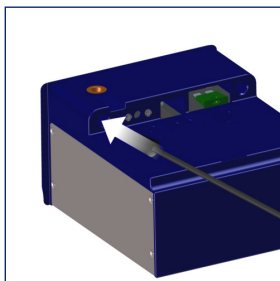
Exchange battery-pack.

- » Press release button and pull out the battery-pack at its handle.

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Connect charger.

- » Keep the plug-in contacts clean!

LED grün = Akku voll

LED green = batteries full

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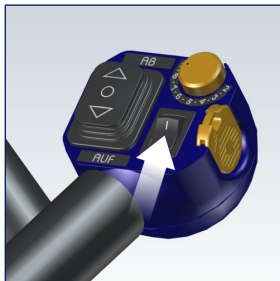
LED orange = batteries are being charged

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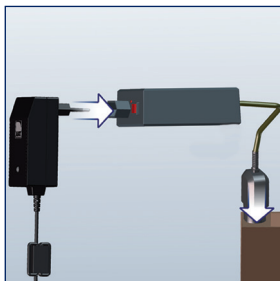
s-max sella ausschalten.

Charge batteries in a vehicle

Turn off the s-max sella.

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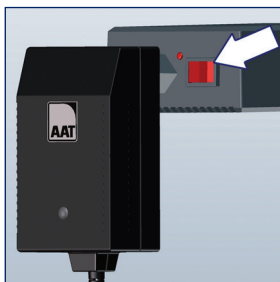
Connect voltage converter.

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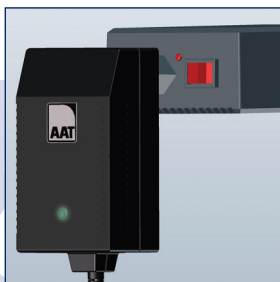
» Warnton = Spannung gering!

Turn on the voltage converter.

» Audio warning = voltage low!

LED grün = Akku voll

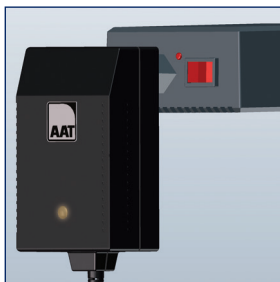
LED green = batteries full

D

GB

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LED orange = batteries are
being charged

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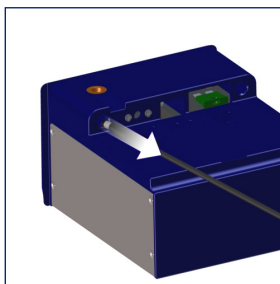
ATTENTION - You may not mend or by-pass a defect fuse! Always replace it with an identical one of the same amperage!

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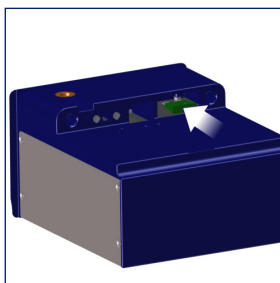
AAT Alber Antriebstechnik GmbH can offer you replacements of the fuses.

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Before you exchange the fuses, please remove the charger from the battery-pack.

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To exchange the fuses, remove the cover and pull out the fuse.

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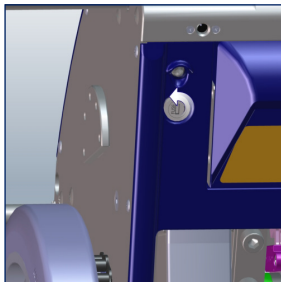
Insert the new one and replace the cover.

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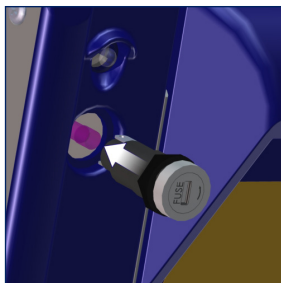
Should your fuses burn out frequently, please contact AAT Alber Antriebstechnik GmbH or your authorized dealer.

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Fuses unit

To change the fuse, push the fuse holder in while turning it counter clockwise.

Pull out the fuse holder and remove the defective fuse. Insert new fuse, turn fuse holder clockwise and fasten.

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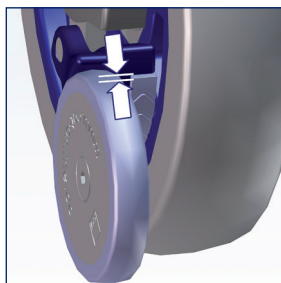
6.3 Check the lining of the brakes

NOTE - Minimum thickness = 1 mm. Should the lining be thinner or damaged, contact the manufacturer or your authorized dealer!

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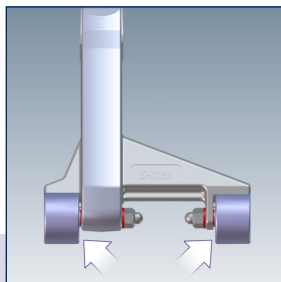
6.4 Check the rollers on the climbing feet

Please also check the rollers on the climbing feet on a regular basis.

If the lining is worn or broken, please have both climbing feet exchanged by AAT Alber Antriebstechnik GmbH or your authorized dealer.

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WARNING - Check safety brakes after cleaning (see chapter 4.5)!

WARNING - Do not smoke while you clean!

NOTE - Clean the safety brakes and the brake drums in the wheel rims weekly or after contamination with spirit. Follow the hazard notes of the spirit!

NOTE - Keep the s-max sella away from sources of ignition!

NOTE - Do not use a high-pressure cleaning device!

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Clean with a moist cloth and household cleaners / disinfectants.

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6.6 Recycling

The s-max sella and its batteries are products of long durability. However, if your s-max sella is old and has reached the end of its lifespan you may return its components to AAT Alber Antriebstechnik GmbH or your authorized dealer to be recycled.

According to the Federal Ministry for the Environment (German BMU) the s-max sella is considered a means of transportation and does not fall under this law concerning the recycling of old electric or electronic devices (ElektroG, §2 paragraph 1, sentence 1 (cross reference, Guideline of the European Parliament 2002/96/EC)).

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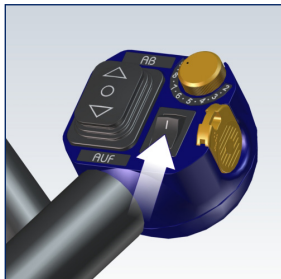
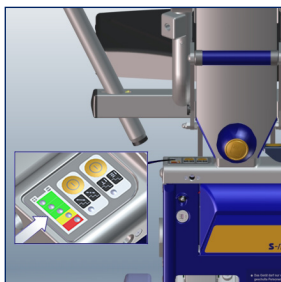
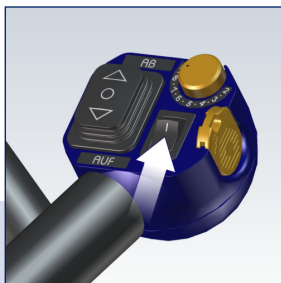
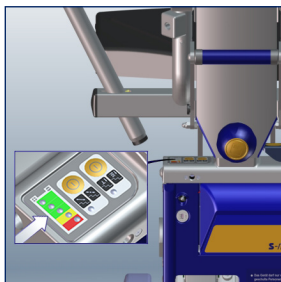
Turn on the s-max sella.

- » LED is blinking - Contact the manufacturer or your authorized dealer!

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Malfuction

LED blin-king	Malfuction
1x	socket is defective
2x	Speed control, UP/DOWN switch, plug-in contacts or supply line are defective
3x	error, circuit board (FET)
4x	error, circuit board (relay)
5x	error, motor
6x	Overloaded
7x	Overvoltage
8x	Malfuction electronics
9x	Malfuction electronics
10x	error, sensor hollow shaft (only if ComfortStep is activated)

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7.2 Trouble shooting

Problem	Check and potential solution
s-max sella cannot be turned on	Is the plug of the helix cable plugged in and locked? No: Plug it in and lock it.
	Is the charger connected to the s-max sella? Yes: Remove the charging cable.
	Are the batteries empty? Yes: Charge the batteries.
	Does the problem still occur? Yes: Contact the manufacturer or your authorized dealer.
s-max sella does not work	Is the safety chip for the engine immobiliser inserted? Yes: Contact the manufacturer or your authorized dealer.
Batteries do not charge	Contact the manufacturer or your authorized dealer.

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8 Warranty and liability

8.1 Warranty

AAT Alber Antriebstechnik GmbH grants a warranty of two years for the s-max sella including all its accessory parts (with the exception of the sealed lead acid batteries). The warranty begins at the date of purchase and covers failure of material and errors in manufacturing.

For the sealed lead acid batteries, always assuming correct maintenance, AAT Alber Antriebstechnik GmbH grants a warranty of one year from the date of purchase.

The warranty becomes void concerning defects or damage which are the result of the following:

- » damage due to excessive strain.
- » damage due to improper handling.
- » violent damage.
- » natural wear and tear (brake lining, gliding strip, climber).
- » improper and insufficient maintenance.
- » unauthorized constructional changes or inappropriate use of the s-max sella or its accessories.
- » using the s-max sella exceeding the total weight approved or the max. approved weight of the occupant
- » improper charging of the sealed lead acid batteries.

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8.2 Liability

The manufacturer AAT Alber Antriebstechnik GmbH, is neither liable nor responsible for potential damage of the s-max sella in case of:

- » improper handling of the s-max sella.
- » repairs, fittings or other works performed on the s-max sella by unauthorized personnel.
- » operation not in accordance with the user manual.
- » extraneous parts added or connected to the s-max sella.
- » parts removed from the s-max sella.
- » operation exceeding the total weight approved or the max. approved weight of the occupant
- » the s-max is not serviced by the manufacturer or authorized personnel every 2 years.
- » persons under age use the s-max sella (except for those minors above 16 operating the s-max under supervision.).
- » operation of the s-max sella with insufficiently charged batteries.

D

GB

Notizen:

Notes:

65

D

GB

Notizen:

Notes:

66

D

GB

Notizen:

Notes:

67

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